

	TEMPORARY RESTORATION					FINAL RESTORATION													
	<i>iphysio Profile Designer</i>	<i>Healing abutment</i>	<i>Direct temporary abutment</i>	<i>Non-rotational temporary abutment</i>	<i>Rotational temporary abutment</i>	<i>Direct clip abutment</i>	<i>Straight or angulated abutment</i>	<i>Customized abutment</i>	<i>Base castable abutment</i>	<i>Customizable abutment</i>	<i>Esthetibase in interfaces non-rotational</i>	<i>Esthetibase in interfaces rotational</i>	<i>Straight Tetra abutment</i>	<i>Angulated Tetra abutment</i>	<i>Straight Plural abutment</i>	<i>Angulated Plural abutment Discontinued</i>	<i>O'ring abutment</i>	<i>Equator abutment</i>	
<b>Tightening torque (N.cm)</b>	15	10	25	25	25	35	25	25	25	25	25	25	35	25	35	25	25	25	
- Abutment																			
-													20	20	25	20			
<b>Restorations</b>																			
- Single-unit	x	x	x	x		x	x	x	x	x	x								
- Plural	x	x	x		x	x	x	x	x	x		x	x	x	x	x			
- Complete		x			x	x	x			x		x	x	x	x	x	x	x	
<b>Prosthesis</b>																			
- Cemented			x	x	x	x	x	x	x	x	x	x							
- Screwed on direct abutment				x	x		x	x	x	x	x	x							
- Screwed on abutment													x	x	x	x			
- Removable																	x	x	
<b>Materials</b>																			
- Titanium	x	x	x	x	x	x	x	x		x	x	x	x	x	x	x	x	x	
- Gold alloy									x										
- Chrome-cobalt									x										