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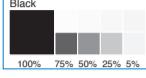
GUIDOR® easy-graft® CRYSTAL+
in situ hardening synthetic bone graft system

Composition

- Biphasic calcium phosphate (60 % hydroxyapatite, 40 % beta-tricalcium phosphate)
- Granules coated with poly(lactide-co-glycolide) (PLGA)
- BioLinker (N-Methyl-2-pyrrolidone solution)

Description

easy-graft CRYSTAL® is a sterile, synthetic bone defect filler. It consists of two components: granules (syringe) and BioLinker (ampulla). After mixing, easy-graft CRYSTAL® is putty-like and can be applied directly from the syringe into the bone defect. In contact with blood or other body fluids, easy-graft CRYSTAL® hardens within minutes to form a solid but porous implant. easy-graft CRYSTAL® is biocompatible and osteo-regenerative. For biphasic calcium phosphate and PLGA in adequate form, no inflammatory or immunoreactions are known. easy-graft CRYSTAL® forms a stable, porous scaffold in the defect area. The granules facilitate the uptake of blood. With the integration of blood, important factors for tissue regeneration are incorporated. In a second step, autogenous tissue can grow into the pore structure. During the initial phase of degradation, easy-graft CRYSTAL® may swell by taking up body fluids. This supports a close contact of the surface with bone tissue. When applied in larger defects, the mix of easy-graft CRYSTAL® and bone tissue is pressed by the surgeon. easy-graft CRYSTAL® contains no animal- or human-derived substances. Biphasic calcium phosphate and PLGA are derived from fully synthetic raw materials.

Indications

easy-graft CRYSTAL® should be used for the filling of mostly unloaded and preferably multi-walled dental or maxillo-facial bone defects. The implantation site should be free of infections and show a granulation tissue. Indications may include:

- Dental periodontal remnants of bone cysts
- Periodontal defects
- Peri-implant defects
- Augmentation of alveolar crest (e.g. Guided Bone Regeneration, GBR)
- Defects after end resection
- Exodontia
- Defects after surgical removal of retained teeth
- Sinus floor elevations
- Defects after removal of autologous bone

Application

Aspects which are essential during application:
easy-graft CRYSTAL® are irregular shaped granules provided in the particle sizes of 450 – 630 µm for smaller defects and 450 – 1000 µm for larger defects.

The amount of easy-graft CRYSTAL® used per patient during one appointment or surgical application should be restricted to a total of 1.8 ml.

For the application of easy-graft CRYSTAL®, refer to the instructions for use table (steps I. to V.).

Dental membranes may be used in combination with easy-graft CRYSTAL® at the discretion of the practitioner.

Contraindications
easy-graft CRYSTAL® must not be used during pregnancy.
easy-graft CRYSTAL® should not be applied in case of the presence of any diseases or therapies which are detrimental to the healing of bone.

No allergic reactions against easy-graft CRYSTAL® or its components are known. However, if an allergic reaction is suspected, the product should not be applied in the affected patient.

Precautions

In order to facilitate the formation of new bone to a maximum extent, easy-graft CRYSTAL® should only be implanted in close contact to well-vascularized surrounding bone tissue (regression of blood clot might be necessary). Care should be taken that easy-graft CRYSTAL® granules are not introduced into surrounding soft tissue, because the granules will only partially decompose.

Before filling periodontal defects, a sufficient pre-treatment is necessary. This should include sufficient plaque control and a successful local treatment of the periodontal lesion (e.g. root planning, debridement).

Possible adverse effects
In general, possible side reactions are comparable to those encountered in autogenous bone or bone graft substitute grafting procedures. These include, but are not limited to: edema, bleeding after surgery, numbness by injured nerve, infection of the soft tissue and/or bone, soft tissue ulceration, incomplete bone formation, lack of bone formation, fracture or inflammation of the newly formed bone.

Allergic or inflammatory reactions to the implant material or its decomposition products have not been recorded.

Interaction with other substances

Not known.

Warnings

easy-graft CRYSTAL® is restricted to dental and maxillo-facial applications and to be sold to, or on the order of a licensed physician or dentist.

easy-graft CRYSTAL® must not be applied to acute infected defects. The implantation site should be free of infections before applying the easy-graft CRYSTAL®.

No do not use easy-graft CRYSTAL® in contact with nerve tissue.

The product has not been clinically tested in children.

Lactation: It is not known whether metabolites of easy-graft CRYSTAL® pass into breast milk.

BioLinker contains N-Methyl-2-pyrrolidone. N-Methyl-2-pyrrolidone is irritant to eyes, respiratory system and skin. Wear gloves and protection goggles when working with BioLinker Change gloves after contact with BioLinker.

After eye contact: In case of contact, immediately flush eyes with copious amounts of water for at least 10 minutes and consult a doctor.

High N-Methyl-2-pyrrolidone content in the BioLinker activation exposure may cause mild irritation, redness, stinging pain and nausea. Therefore the amount of easy-graft CRYSTAL® used per patient during one appointment or surgical application should be restricted to a total of 1.8 ml.

Do not use if package is opened or damaged, or if the expiration date has been exceeded.

easy-graft CRYSTAL® cannot be reused. Discard any unused material. Sterility and function are guaranteed in the event of a reuse.

Properties
easy-graft CRYSTAL® consists of biphasic calcium phosphate, PLGA and BioLinker. Resorption proceeds via three steps: 1) The BioLinker is metabolized and excreted within days mainly through the urine; 2) The absorbable polymer (PLGA) is cleaved by hydrolysis. The degradation products are excreted. 3) The biphasic calcium phosphate is partially resorbed by biological mechanism and direct dissolution. A part of the biphasic calcium phosphate remains embedded in autogenous tissue for prolonged periods of time in order to preserve the volume.

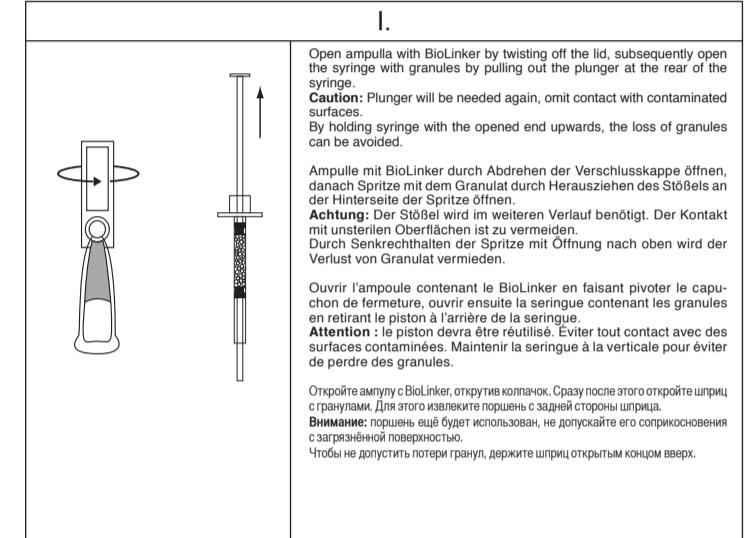
easy-graft CRYSTAL® supports bone regeneration and bone formation. The application of the material does not guarantee a treatment success. The success of the therapy depends on many factors such as surgical technique and habits, age and the regeneration potential of the patient.

Miscellaneous
easy-graft CRYSTAL® is gamma sterilized and intended for single application.

Storage instructions
easy-graft CRYSTAL® must be stored in its original packaging. Store between 5°C/41°F to 25°C/77°F. Keep away from sunlight.

Manufacturer
Degradable Solutions AG, Wagistrasse 23, 8952 Schlieren / Zurich, Switzerland

Instructions for use / Gebrauchsanweisung Mode d'emploi / Инструкция по применению

I.

Open ampulla with BioLinker by twisting off the lid, subsequently open the syringe with granules by pulling out the plunger at the rear of the syringe.

Caution: Plunger will be needed again, omit contact with contaminated surfaces.

By holding syringe with the opened end upwards, the loss of granules can be avoided.

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